

 **JENSEN**  
DENTAL

# QUICK START GUIDE deutsch

For a perfect start.





## Preparation | Instructions for use

### Preparation


- Before applying the material, the surface must be clean and free of surface contamination.
- The surface can be sandblasted with 50/110 µm and 1.5 bar pressure.

### Instructions for use

- Do not pour surplus liquid out of the container. Mix it back into a paste.
- Mix MiYO Color, MiYO Structure and Glaze Paste thoroughly with a metal free spatula before each use.
- The pastes must not come into contact with water.
- Always use a clean and dry brush.

### Storage and disposal

- Store in a dry place.
- Small quantities can be deposited in household waste.

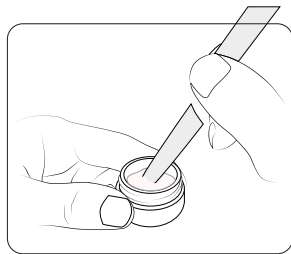

**Mix thoroughly with a metal-free spatula.**



## Firing chart

Firing parameter*	
Start temperature [°C]	450
Dry time [min]	3
Closing time [min]	4
Heat rate [°C / min]	45
Vacuum start [°C]	580
End temperature [°C]	710-720
Vacuum end [°C]	710-720
Holding time [min]	1
Opening time [min]	1

\* The firing temperatures are standard values and may vary depending on the type of furnace.



## Downloads

The Safety Data Sheet and the Quick Start Guide are available for download at:

[www.jensendental.de/downloads](http://www.jensendental.de/downloads)



## Webshop

24/7 you can order in our order in our webshop.

<https://shop.jensendental.de/>



## Support

**General requests:**

[info@jensendental.de](mailto:info@jensendental.de)

+49 7123 92260

**Technical support:**

[support@jensendental.de](mailto:support@jensendental.de)



**Sales and Technical Support Europe:**

Jensen Dental GmbH  
Gustav-Werner-Straße 1  
72555 Metzingen, Germany  
0800-857 32 30 | +49 7123-92260  
info@jensendental.de  
www.jensendental.de | www.miyoworld.eu

**Manufactured by:**

Chemichl AG  
Landstrasse 114  
9490 Vaduz, Liechtenstein  
info@chemichl.com  
www.chemichl.com



**GAN: 03 | 2207**  
English