MONOMASSA

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Protocols using the Vertysystem® technique

Guide for creating resin temporaries with the Monomassa technique.

Issuing a manual for creating temporary dentures nowadays might seem like looking at the past, not at the future.

But if we think about what we want to achieve in terms of quality in our prosthetic the technique we will explore cannot be considered obsolete.

In fact, with the Vertysystem technique we can perfectly replicate what we have achieved in the diagnostic wax and / or wax-ups.

All this with the possibility of obtaining the provisional without occlusal extensions and with the maximum precision in the marginal closures, thus improving the quality of the manufacture.

Keeping in mind also the economic side, we have created some Monomassa resins that have perfectly calibrated colours, making traditional techniques easier and quicker.

This manual is intended as a step by step guide to realise a provisional denture in an easy and repeatable way.

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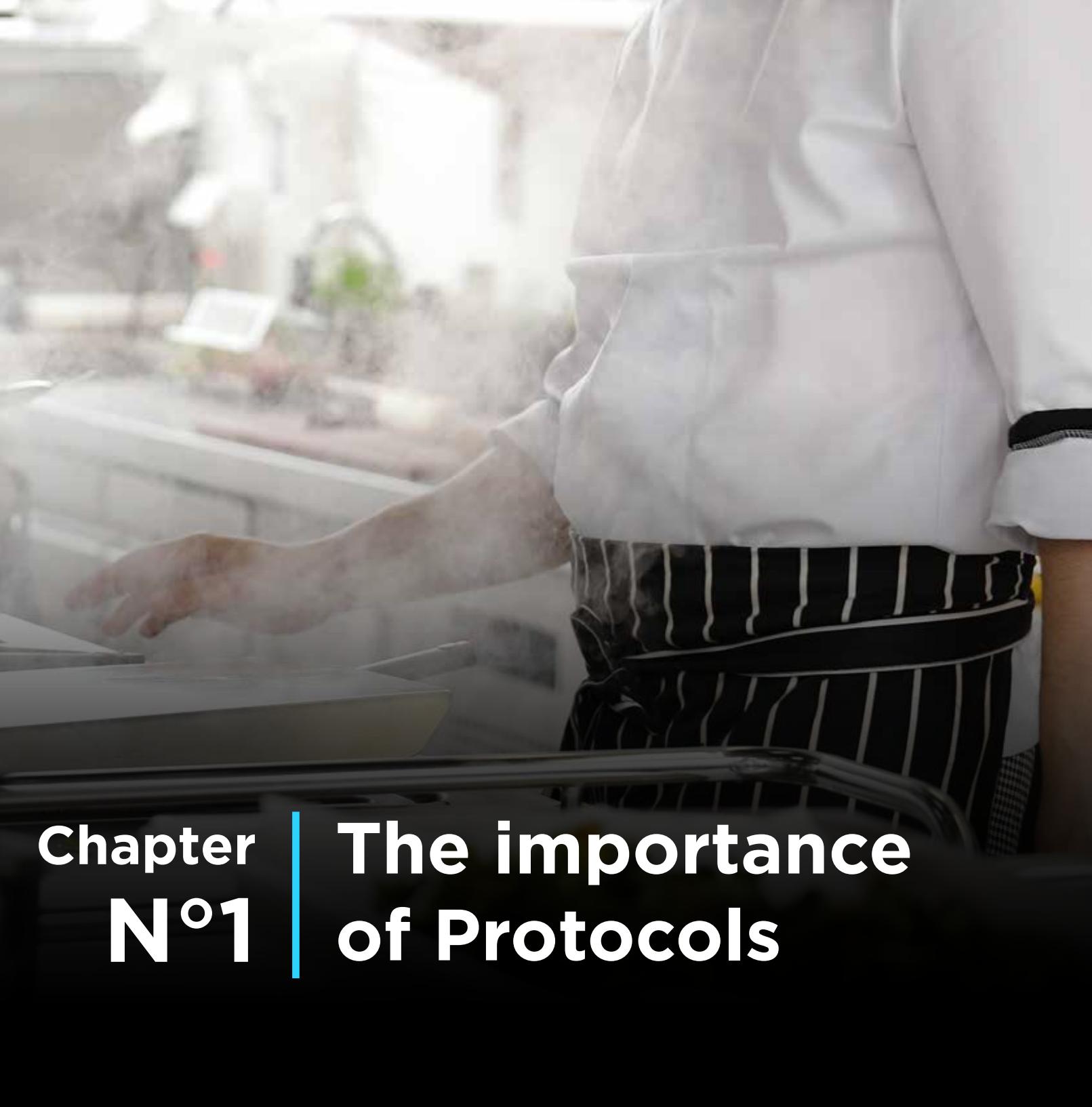
*Claudio Nannini was born in Modena on Aug. 21, 1961.
He runs his own laboratory in Modena with four employees: Luca, Roberto, Elvir e Luisa.
He's ANTLO partner since 1985 and speaker since 1988, and is a member of the ANTLO. Speakers Union.
He is part of the scientific committees of the journals "Dental Dialogue" and "Quintessenza".
He participated, as a speaker, at numerous national and international events and in particular he ran the pre-conference course at the Amici di Brugg Convention in 2005, with Dr. Cuman and Prof. Slavicek.
He has written several articles in national and international journals such as: Il Laboratorio Odontotecnico, RTD - SICED, Dental Dialogue, Team Work.
Alongside his colleague Baldin and doctors Cuman and Masnata, he wrote the book "The total denture according Slavicek" published by Team Work Media.
In 2015 Team Work media published his latest book, "AESTHETIC and FUNCTION".
He's been holding courses for DOCTORS and TECHNICIANS on Gnathology / Byte / PRESS Techniques with lithium silicate for years.
He gave a very important support to a.gree ltd for the development of the Monomassa resin, documenting a real case for this book.*

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A black and white photograph of a chef in a professional kitchen. The chef is wearing a white chef's coat and a dark apron with white vertical stripes. They are leaning over a stainless steel stove, their hands positioned over a large metal pan. Steam or smoke is rising from the pan, indicating it is hot. In the background, there are shelves with various kitchen items and ingredients. The lighting is dramatic, with strong highlights and shadows.

Chapter N°1 | The importance of Protocols

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Models mounted on the articulator, indispensable tool for the realization of prosthetic restorations when we need to build a new front rail.

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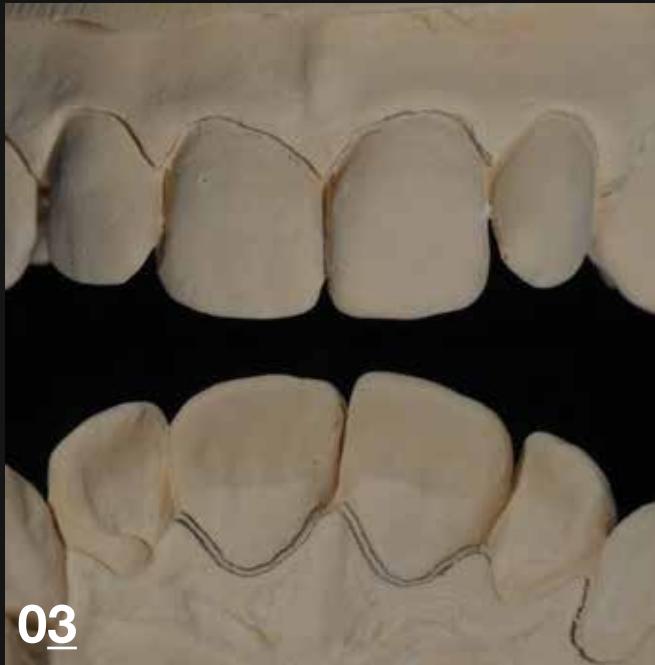
Upper model in occlusal view. Details of the front and of the upper left side, where the provisional dentures will be created.

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Pencil identification of the closure margins of the pre-burrying provisional denture in cervical areas of the teeth involved. The gingival sulcus is the most defines, to allow the technician to have a greater thickness in the execution of the prosthetic and the clinician to have a margin of safety during the stages of preparation of the teeth as they will be sure of the fit.

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Additional diagnostic wax on the existing dental anatomy. Increasing the anatomy of the model, in the cervical areas, using the wax is important, in order to guarantee the thickness necessary to correctly realise the temporaries dentures.

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When we perform the reconstructions of the front guide (protrusion and laterality) it is critical to use an articulator to simulate and then reproduce, as close as possible, the position of the models in the mouth and thus the closure in centric and eccentric movements of patient.

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View of the model in intercuspal on both the vestibular side and the palatal side. In this position it is important to create a situation of stability in the centric so as to ensure a univocal and defined position. This will be of considerable help to the clinician when it is time to re-line the provisional dentures in the mouth as they will be certain to perfectly identify the position of maximum intercuspidation in the mouth.

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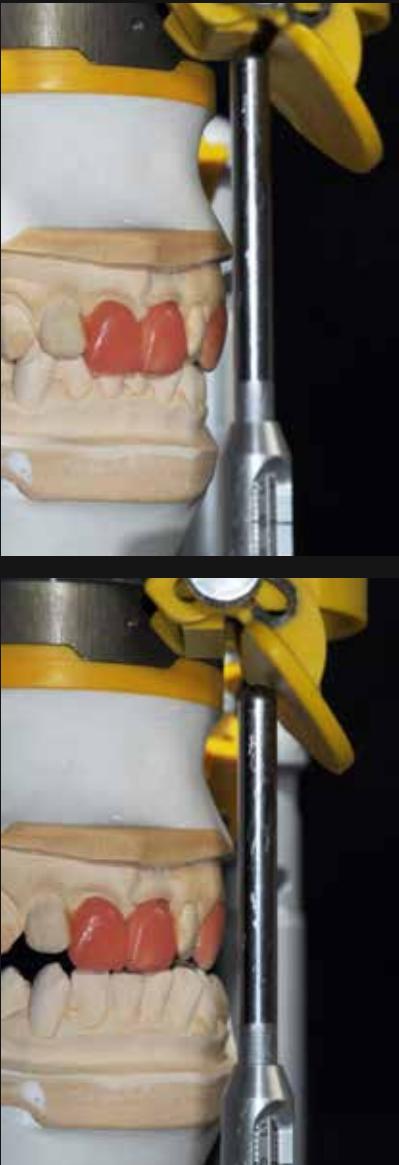
Using the accessories supplied with the SAM articulator, like the adjustable incisal table and the protrusion inserts, we will be able to determine the angle of the front guide and the position of maximum protrusion. To adjust the table, we will tilt the incisal plate until the posterior teeth are reachable, and to determine the overjet from the centric point on the mesial lobes of the upper incisors we will use the protrusion inserts on both sides, until we reach the desired position.

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Likewise, for lateral movement we will turn the incisal table by 60° towards the area in which we want to perform the disclusion template. To determine the length of the tooth template, we will use the protrusion inserts on one side of the articulator, while tilting the incisal plate to adjust its inclination, until the posterior areas are reachable.

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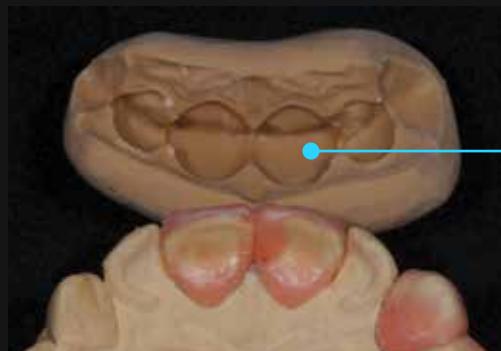
In this case as well, it is clear that the wax guide of the palatal area of the canine tooth perfectly matches the inclination of the adjustable incisal table. In the other pictures you can see the disclusion of the posterior teeth both on the mediotrusion side and on the laterotrusion side.

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You can now proceed to the pre-burring of the teeth, preparing templates that allow you to check the spaces of plaster preparation in an optimal way.

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*At this point you can prepare the **Furbo** flask to create the temporary dentures. You should use the silicones best suited to the method (**Vertys Easy Putty**, **Vertys Orange Plus** and **Vertys Precision**) to achieve the best possible accuracy and definition. In this case the connection between the base of the flask and the working model has been prepared using silicone and putting notches under the model to ensure a precise and safe positioning, but we can use **Vertys Split Cast** or **Vertys Artic Link M** to achieve the same final result.*

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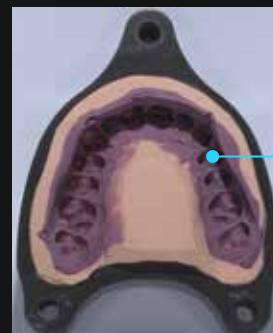
*The spacer nuts are important, as they allow you to realise the project with greater ergonomics (less waste of silicone), and so is the clamping system, using the dynamometric screw **Vertys Dynamo Block**, which allows you to have a uniform and calibrated tightening of the **Furbo** flask prior to injecting the resin.*

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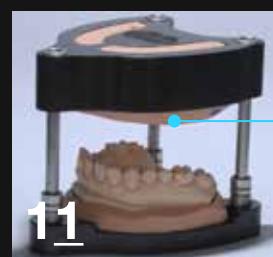
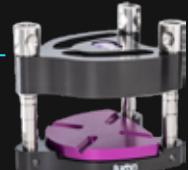
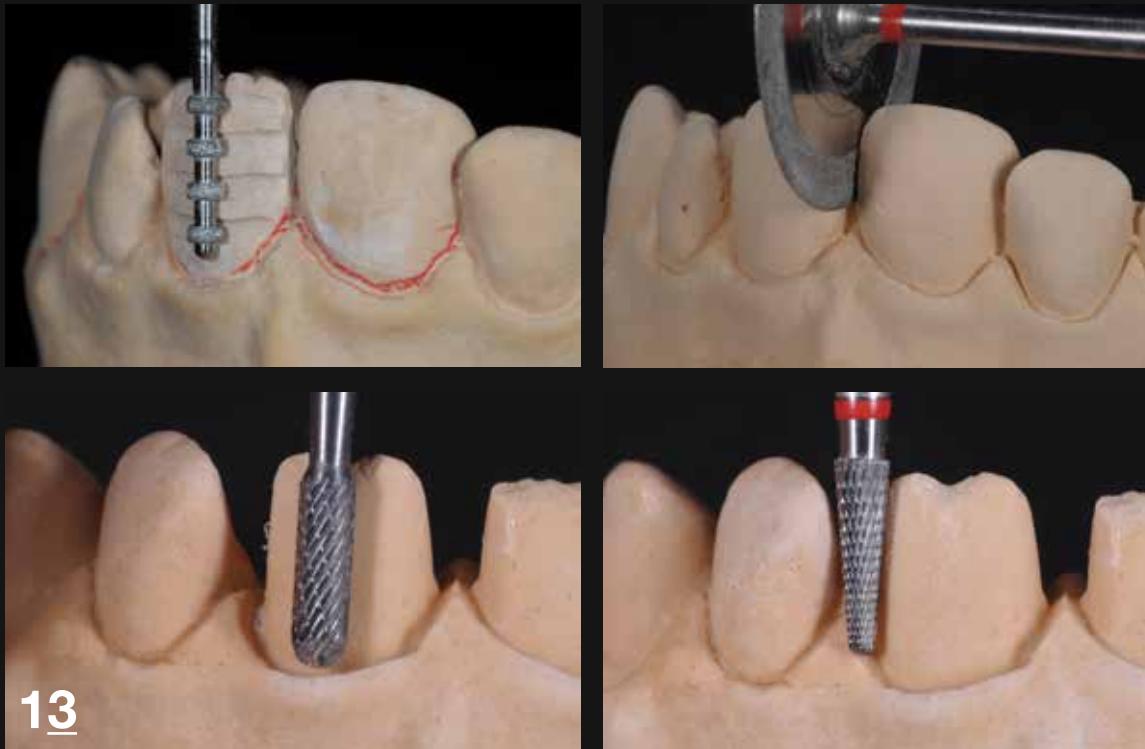
Here are the basic steps to get a uniform result and a calibrated control of the spaces. These are the burs that we use for the preparations, paying attention not to touch the proximal teeth and to avoid the last millimeter before the cervical margin; in these areas we will simply eliminate any undercuts. Furthermore, it is important not to go too deep in the proximal areas, as it would make inserting the denture in the mouth problematic for the clinician.

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Check the spaces with appropriately cut templates.

Note how it was important to augment the anatomy of the cervical margins with wax. In this way we will have great closure, and slightly abundant provisional dentures that the clinician won't have any difficulty inserting in the mouth.

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Here are finished preparations; you may notice that the cervical margins defined by pencil remained intact and that means we did not go beyond the limits that we had determined earlier. Also important are the preparations of the canals and areas that need to be clean. It's better to make them slightly more compressive, keeping the design of the peaks of the papillae, so that the clinician can assess and eventually discharge in the control phase.

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Logically the spaces are checked in relation to the opposing bite. In these images the preparations of the pontic areas are clearly visible to preserve the papillae. At the end of this phase, we recommend applying a coat of hardener (Margidur) to the prepared coronal areas. This way the crowns in polymerized resin will be smooth and clean.

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Model inserted into the flask with the preparations made.

*Special silicone relined with **Vertys Precision** and access holes (input and output) for resin injection.*

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Use Verty's Latex Pink, an indispensable condition to get a perfect insulation resin / plaster and to avoid breaking or chipping the plaster after curing the resin. It should be applied in a thin and uniform layer when the work is done, it comes off perfectly, leaving the internal surfaces of the resins smooth and clean.

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Tempplus Artist Kit

200-2391	Tempplus Glaze
200-2451	Artist White
200-2456	Artist Mustard
200-2457	Artist Brown

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Phases of resin injection through the fitting holes with the aid of a syringe.

It's important to pay attention to the closing of the flask with the torque wrench and the consistency of the resin, that is to be injected when in the plastic state and not the liquid state. In this way we can avoid bubbles and the final structure of the resin is more compact and without porosity.

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Here is how the resin looks after curing, with the input/output pins still present. After small finishing touches like removing the pins, the work is perfectly placed on the working model. The result reflects the anatomy of the model and interior walls are uniquely clean.

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Checks on the accuracy of the marginal closures of the resins made with Monomassa resin for pre-burring provisional dentures.

The final product is the result of a methodology and a work protocol that if executed properly ensure this result.

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Finished and polished works in vertys Monomassa. You may add small characterizations to individualize the temporary.

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Born in Parma in 1975, he attended the Professional Institute "Galvani" in Reggio Emilia obtaining a dental degree in 1994. From 1996 he worked at a major laboratory in Parma part of the - Dental Excellence (International Laboratory Group) circuit,

collaborating and developing the technique of double crowns on AGC - Wieland systems.

In 2012 he started running the laboratory in Guastalla (Reggio Emilia) where he continued specializing in fixed prosthodontics with a focus on functionality and aesthetics.

He attended courses with internationally renowned speakers such as: Joechen Peters - Gerard Ubassy - Nondas - Klaus Mütterthies - Claudio Nannini.

He collaborates with leading companies in the sector including A.GREE - VERTYSYSTEM.

He became an opinion leader and contributor to the OT-IMPLANT (RHEIN-83).

He publishes articles on national and international trade magazines.

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Esteemed Colleague,
I have started using the Monomassa Vertys Tempplus too, because it allows for a simpler, more economical way of working, while still obtaining a great result aesthetically.
I use the Monomassa to create clones of Full Dentures, Implant Bridges, Immediate Dental Implants, Posterior Provisional Dentures, Pre-extraction Dentures and so on.
I think this product is absolutely genius, because the transparency comes out as the tooth thins out, without ever losing the chromatic value of the dentine.
Using characterizing surface colours, one can further enhance the qualities this material possesses, as it already contains a mixture of coloured dentine, transparencies and enamels.
Another great advantage is the possibility of further customisation of the tooth, by adding some Transparent and/or Intensive Orange enamel to the Monomassa.
Those who, like me, already use the Vertys Tempplus temporary resin can use the same monomer, because the liquid is one and the same.
In this manual, I have reported my daily experience.

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01

Clone in resina Vertys Prothesis Excel - *Vertys Prothesis Excel resin clone*

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Ideal for any kind of temporary, but especially for provisional veneers.

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To create anterior provisional dentures with a medium-thick rim, mix in the material 10% of Vertys Templus Transparent enamel.

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To enhance the transparency and add depth to reconstructions such as implant bridges, full dentures, and pre-extraction dentures, mix in the material 30% of Vertyx Tempplus Transparent enamel.

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The chroma of all four Monomassa colours can be enhanced using Vertyx Tempplus Intensive Orange 200-2389.

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(in Classic Veined Pink customised with Character Dark)

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ENG

Vertys Templus Line (temporary resins) is now enriched by a new project: "Monomassa" Vertys Templus by Claudio Nannini.

All the resins "Monomassa" Vertys Templus by Claudio Nannini are suited for the traditional Vertys Templus Line. the "Monomassa" Vertys Templus by Claudio Nannini is composed by 4 categories:

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Light color indicated for aesthetic temporary. This color was designed also for the clinicians that want to realize direct temporary. Starting from this light base you can obtain the color you desire, adding the intensive Orange. The base of the "Monomassa" MM1 correspond to pattern A1 (shade guide VITA).

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The right mix of dentines, enamels and intensives has allowed that this "Monomassa" was perfectly designed for 90% of all temporaries workings. The base of this "Monomassa" MM2 is a mix between pattern A2/A3 (shade guide VITA). Using the intensive Orange you can reach the color you want.

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Intense color whose base is a pattern A3,5 (shade guide VITA). Hot tonality. Indicated for A3,5/A4 workings. It is possible to further intensify the color with the intensive Orange.

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That color is similar to pattern D3 (shade guide VITA) but with more transparency. Instead, in thicker areas, this "Monomassa" is more saturated. The results is a beautiful resin, equilibrated in transparency and colored by some grey reflexes. The base of "Monomassa" MMG1 is the pattern D3.

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